AIRFLOW

STATION & DEVICE SUMMARY

EN	STATION & DEVICE INSTALLATION AND CONNECTIONS	2	LV	STACIJAS UN IERĪCES UZSTĀDĪŠANA UN SAVIENOŠANA	17
BG	МОНТАЖ И СВЪРЗВАНЕ НА СТАНЦИЯ И УСТРОЙСТВО	3	MS	PEMASANGAN DAN SAMBUNGAN STESEN DAN PERANTI	18
CS	INSTALACE A PŘIPOJENÍ STANICE A PŘÍSTROJE	4	NL	INSTALLATIE EN CONNECTIES VAN BASISSTATION & APPARAAT	19
DA	INSTALLATION OG TILSLUTNING AF STATIONEN OG Udstyret	5	NO	STASJON OG ENHET INSTALLASJON OG KOBLINGER	20
DE	STATIONS- UND GERÄTEINSTALLATION UND ANSCHLÜSSE	6	PL	MONTAŻ I PODŁĄCZANIE STACJI I URZĄDZENIA	21
EL	ΕΓΚΑΤΑΣΤΑΣΗ ΚΑΙ ΣΥΝΔΕΣΕΙΣ ΣΤΑΘΜΟΥ ΚΑΙ ΣΥΣΚΕΥΗΣ	7	PT	INSTALAÇÃO E LIGAÇÕES DA ESTAÇÃO E DO DISPOSITIVO	22
ES	INSTALACIÓN Y CONEXIONES DE LA ESTACIÓN Y EL DISPOSITIVO	8	RO	STAȚIE & DISPOZITIV INSTALARE ȘI CONEXIUNI	23
ET	JAAM JA SEADE PAIGALDAMINE JA ÜHENDUSED	9	RU	УСТАНОВКА И ПОДКЛЮЧЕНИЕ СТАНЦИЙ И АППАРАТА	24
FI	STATIONIN JA LAITTEEN ASENNUS JA LIITÄNNÄT	10	SK	INŠTALÁCIA A PRIPOJENIA STANICE A ZARIADENIA	25
FR	INSTALLATION ET RACCORDEMENTS DE LA STATION ET DE L'APPAREIL	11	SL	POSTAJA IN NAPRAVA – NAMESTITEV IN PRIKLJUČKI	26
HR	POSTAJA I UREĐAJ – POSTAVLJANJE I PRIKLJUČCI	12	SV	INSTALLATION OCH ANSLUTNINGAR FÖR STATION OCH ENHET	27
HU	ÁLLOMÁS ÉS ESZKÖZ ÜZEMBE HELYEZÉS ÉS CSATLAKOZTATÁS	13	TH	สัดตั้งและและอุปกรณ์ การติดตั้ง และเชื่อมต่อ	28
ID	STATION & KONEKSI DAN PEMASANGAN PERANGKAT	14	TR	İSTASYON VE CİHAZ KURULUM VE BAĞLANTILAR	29
IT	INSTALLAZIONE E COLLEGAMENTI DELLA UNITÀ E DEL DISPOSITIVO	15	VI	LẤP ĐẶT VÀ KẾT NỐI TRẠM VÀ THIẾT BỊ	30
LT	STOTELĖS IR PRIETAISO ĮRENGIMAS IR JUNGTYS	16			

AIRFLOW

AIRFLOW® Prophylaxis Master

AIRFLOW® One

STATION & DEVICE INSTALLATION AND CONNECTIONS

Compressed air source

An oil-free compressor with dryer is highly recommended.

Stable air source - no moisture in the air lines.

Pressure: 4.5 to 7 bar (65 to 101 psi). Ideally 6 bar (87 psi)

Humidity max: 1.032 g/m³ - Oil filtered max: 0.1 mg/m³ - Filtration: max. 1 µm

Minimum flow rate 20 NI/min at 4.5 bar (65 psi)

Water source

Drinking water

Pressure: 2 to 5 bar (30 to 72 psi)

Temperature: 10 to 30 °C

Salinity max: 0.2%

Power source

100-240V AC ~ 50/60 hz, 4A max

⚠ TO AVOID the risk of electric shock, this equipment must only be connected to a mains supply with protective earth/grounding. This device uses a Class-I insulating system that requires protective earth.

Depending on the dental unit, use the correct air and water connectors. They are not provided by EMS¹. Contact your field service technician to obtain and install them.

⚠ MAKE SURE not to switch the air and water connections: the EMS warranty does not cover damages in case of unit flooding due to incorrect connections.

The air hose (EH-142^{*}) is identified by white tracers, the water hose (EG-110^{*}) by blue tracers and a "H2O" sticker.

Air hose EH-142^{*}

Water hose EG-110^{*}

Ideally, the air/water connections and the power output should be located in close proximity at the base of the dental unit (see photo).

Make sure the power, air and water hoses do not present any risk of falling.

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EMS UK LTD SUITE 2
E.M.S. ELECTRO MEDICAL SYSTEMS TRADING (SHANGHAI) CO., LTD.
EMS NORDIC AB
EMS ELECTRO MEDICAL SYSTEMS KOREA LTD.